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Supplementary sheet to the user manual

Warning:

Please check before using this mask whether it is suitable for your purpose. This mask must not be used in case of damage to the mask itself or to the packaging.

If you do not achieve a satisfactory fit, never enter contaminated areas! Contact your supervisor or a specialist nearby.

The marking „NR“ means „Non Reusable“ and states that the mask may only be used for a maximum of one work shift (8 hours) and must then be disposed of professionally. It is not reusable.

Class FFP2 respirators are suitable for working environments in which harmful substances are present in the breathing air. They must capture at least 94 % of the particles in the air up to a size of 0.6 µm and may be used if the working limit value does not exceed 10 times the concentration.

This mask does not provide oxygen and must not be used in atmospheric environments with less than 19.5 % oxygen. Do not use this mask in hazardous areas.

Do not use this mask with beards or other facial hair that could impede contact between the face and the edge of the mask. Only use the mask with healthy skin.

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FFP2 NR Respirator mask / Medical face mask

Filtering half-mask for protection against aerosols and particles and for use in infection control

TechMed Mask Art. 90101

FFP2 NR EN 149:2001+A1:2009 CE 0197

Typ IIR DIN/EN 14683:2019-10 CE

Product size folded 16 x 10,5 cm – store dry

Instructions for use

1. Thoroughly clean and disinfect your hands before applying the mask. Remove the mask from its packaging and gently pull the straps apart to unfold the mask. Check the mask for defects or damage.

5. Pull the mask down over mouth and chin. Take a deep breath. The mask must cover the nose, mouth and chin completely and fit snugly. If the mask fits correctly, the filter material will move when you breathe in.

2. Hold the mask in front of your face with the inner surface facing upwards and place a ribbon around one ear at the right and left side.

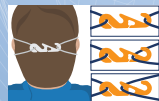
6. Avoid touching the front of the mask. Only touch it by the straps. In case of wetness and moisture, please put on a new mask.

3. Press with your fingers on the moulded frame and adjust it to the nose so that the mask fits tightly.

7. The recommended wearing time of the mask is one shift (approx. 8 hours). Throw the mask in the bin after use. Then wash and disinfect your hands again.

4. The length of the ear loops can be adjusted behind the head using the clips provided.

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Manufacturer

LT TECHNOLOGIES
service is not just an idea!

LT technologies GmbH & Co. KG | Langes Gewand 7
78604 Rietheim-Weilheim | Germany
Fon: +49 (0)7461 9 66 20 - 0
info@lt-technologies.de | www.lt-technologies.de

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The EU declaration of conformity on the website www.lt-technologies.de can be viewed and is available.

2022-08

2024-08